

## QUICK-START GUIDE

### • Connect Accessories

- Insert blue paring key into the monitor. Light will blink 6 times.
- Insert key into each pad's transmitter unit until key light flashes quickly
- Remove key & the monitor will audibly confirm connection
- Store key in the monitor after all pads & accessories are connected
- Test a pad by applying pressure for 3 seconds, then release & listen for alarm. Press reset to silence the alarm.
- Place pad under patient's buttocks according to the diagram on the pad  
*Note, only one of each type of accessory can be connected to the monitor.*

### • Record a Personalized Message

- Press & hold **REC** button
- When you see the green light begin recording your message (up to 13 seconds)
- Release **REC** button when finished
- To delete a message, quickly press & release **REC** button

### • Select Local Alarm Mode & Tone (Back of Monitor)

- **A** - Alarm locally for both incontinence & pressure
- **B** - Alarm locally for pressure only
- **C** - Alarm locally for incontinence only
- **D** - No local alarm
- Use **1-2-3-4** switch to choose a local alarm tone

*Note, alarms will always sound at nurse call station regardless of alarm mode setting.*

### • Button Functions

Hold pressure alarm for 30 seconds	<b>HOLD</b>	STATUS	RESET	Click once
Hold incontinence alarm for 5 minutes	<b>HOLD</b>	STATUS	RESET	Click once
Visual confirmation of connections	STATUS	<b>RESET</b>	Click once	
Audible confirmation of connections	STATUS	<b>RESET</b>	Click twice	
To confirm tone & mode settings	STATUS	<b>RESET</b>	Press & hold	
Playback recorded message	STATUS	<b>RESET</b>	Press & hold	
Silence alarm or pre-recorded message	STATUS	<b>RESET</b>	Click once	
Adjust volume	<b>HOLD</b>	STATUS	<b>RESET</b>	Press & hold
Disconnect lost pad	STATUS	<b>RESET</b>	Press & hold	
Reset for new patient (Nurse call will stay connected)	<b>HOLD</b>	STATUS	RESET	Press & hold
Factory reset (Clears everything)	<b>HOLD</b>	STATUS	<b>RESET</b>	Press & hold

To watch our quick-start video, visit [parasolmed.com](http://parasolmed.com)